


Preanalytics: Important directions for best quality

- Carefully sway every tube several times after taking the blood
- Please label the tubes for clear identification!
- **Do not refrigerate or centrifuge any tube!**
- Store the tubes at room temperature and protect them from sunlight.
- Please ship the tubes only in our special test kit.
- Logistics:
 1. Either use our established Logistics Express Service
 2. Or ensure delivery within a maximum of 72 hours

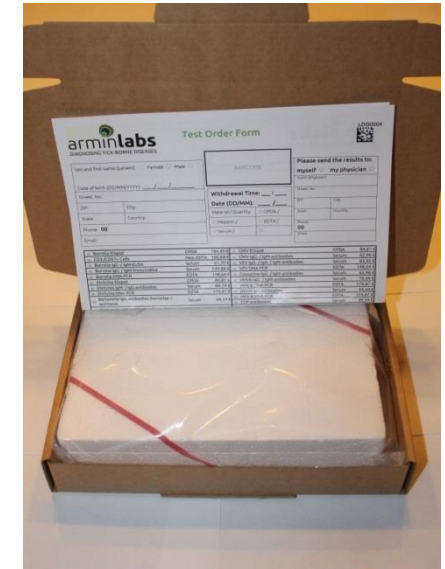

As a rule, your results should be available to you within 1 week after receipt of the sample with standard tests, and within 2-3 weeks with third-party services.

If you have any questions, complaints or require further information, please do not hesitate to contact us!

Phone: 0049 821 780 931 50

Mail: info@arminlabs.com

Internet: www.arminlabs.com


arminlabs
Medicum Bad Aibling MVZ GmbH

Created: 01.02.2024	Checked: 01.02.2024	Release: 06.02.2024
Name: Neugebauer Veronika	Name: Härtl Marina	Name: Neugebauer Veronika
Revision: 1	Revision data: 01.02.2024	Change: General Revision

Blood Drawing – Using the right tubes:



Antibodies: 1 x Serum tube

Stability: Up to 14 days



EliSpot: 3 x ACD/CPDA tube

(please draw all 3 tubes if possible)

Stability: Up to 3 days



CD3-/57+ Cells: 1 x Heparin tube + 1 x EDTA tube

Stability: Up to 3 days



PCR: 1 x EDTA tube

Stability: Up to 5 days